

SRS II

Self-Retaining Bicanaliculus Intubation Set II


Now Preloaded on Introducers

INDICATION

For Horizontal Lacrimal
Duct Obstruction and
Punctal Stenosis



www.FCI-Ophthalmics.com

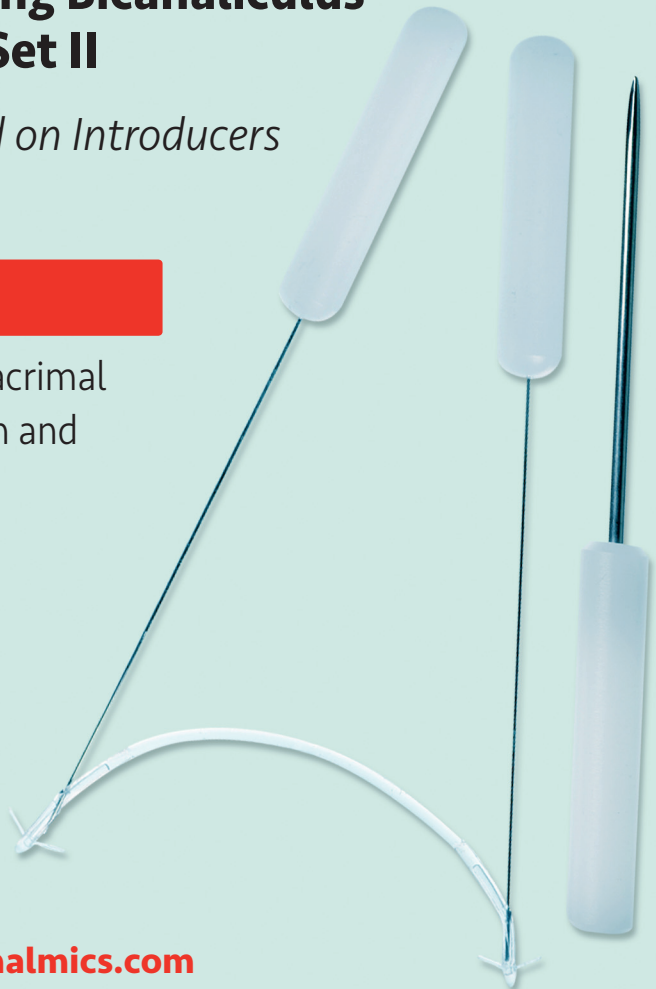

SRS II

SELF-RETAINING BICANALICULUS INTUBATION SET II (SRS II)

The SRS II enables intubation of the upper and lower canaliculi without having to enter through the nasal passageway. The 0.64 mm wide tube is made of medical grade silicone and is available in three lengths – 25 mm, 30 mm and 35 mm. Each end of the tube has an anchor-shaped head with two flexible winglets that fold inward during insertion through the punctum. After passage through the common canaliculus, the winglets open in the lacrimal sac for secure fixation. A centrally placed marking on the tube acts as a reference point and allows verification of proper stent positioning.

FEATURES & BENEFITS

QUICK & SIMPLE

- In-office procedure using anesthetic eye drops
- Less invasive
- Virtually painless for the patient


EASY & SECURE

- Flexible winglet design enables secure fixation
- Guides give rigidity to the tube and facilitate placement of the stent
- Central mark offers control of proper stent positioning

CONVENIENT

- Available in three lengths – 25 mm, 30 mm and 35 mm
- Includes disposable dilator

Disposable Sizer (S1-1289) sold separately

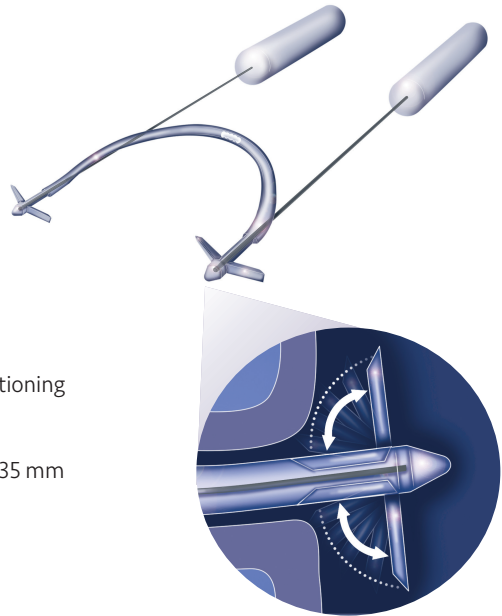

SRS II

S1-1390

25 mm
Box of 1
sterile

S1-1391

30 mm
Box of 1
sterile

S1-1392

35 mm
Box of 1
sterile

S1-1289u

Disposable Sizers
Box of 5
sterile

Insertion Through Punctum:

Winglets Fold Inward

**Passage Through
Canaliculus:** Winglets
Open in Lacrimal Sac



FCI OPHTHALMICS: 20 Winter Street • Pembroke, MA 02359

EMAIL: info@fci-ophthalmics.com

TELEPHONE: 800.932.4202 • 781.826.9060

FAX: 781.826.9062

www.FCI-Ophthalmics.com